Particulate Filter, P100

User Instructions

IMPORTANT: Keep these *User Instructions* for reference.

For use only with products listed below according to NIOSH approval labels:

- 3M™ Half Facepiece Respirator 5000 Series, or
- 3M™ Half Facepiece Respirator 6000 Series, or
- 3M™ Half Facepiece Respirator 7000 Series, or
- 3M™ Ultimate FX Full Facepiece Reusable Respirator FF-400 Series, or
- 3M™ Ultimate AV-632 Bayonet Adapter with 3M™ Scott™ AV-3000 HT Facepieces and 3M™ Scott™ AV-3000 with SureSeal Facepieces.

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98-0060-0062-8, 6 34-8726-3943-9

This respirator filter has dual approval as a United States (US) National Institute for Occupational Safety and Health (NIOSH) P100 particulate filter and as a Mexican Ministry of Labor approved P3 SL particulate filter. Information specific to local standards is provided where applicable. All other information is common to both standards.

Refer to facepiece *User Instructions* for installation.

⚠ WARNING This filter helps protect against certain airborne contaminants. **Misuse may result in sickness or death.** For correct use, consult supervisor and *User Instructions*, or call 3M in U.S.A. at 1-800-243-4630. In Canada, call Technical Service at 1-800-267-4414. In Brazil, contact: 0800-0132333.

Use filter before expiration date. 

IMPORTANT

Before use, the wearer must read and understand all *User Instructions* for the respiratory system according to NIOSH approval labels. Keep these *User Instructions* for reference.

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7093

Use For

- Certain solids such as those from processing minerals, sanding and grinding.
- Certain liquid or oil based particles from sprays that do not also emit vapors.
- Certain metal fumes produced from welding, brazing, cutting and other operations involving heating of metals.
- Radioactive materials such as uranium and plutonium.
- Asbestos.

For additional information on 3M use recommendations please consult the 3M Respirator Selection Guide found on the 3M Personal Safety Division (PSD) website at www.3m.com/respiratorselector or call 1-800-243-4630 in U.S.A., in Canada call 1-800-267-4414. In Brazil, contact: 0800-0132333.

Do Not Use For

Do not use for gases and vapors when concentrations exceed the OSHA PEL, or applicable government regulations, whichever is lower, including those present in paint spraying operations, unless combined with approved chemical cartridges. Do not use for sandblasting.

Biological Particles

This product, when used in combination with 3M products according to NIOSH approval labels, can help reduce inhalation exposures to certain airborne biological particles (e.g. mold, *Bacillus anthracis*, avian influenza, *Mycobacterium tuberculosis*, etc.) but cannot eliminate the risk of contracting infection, illness or disease. OSHA and other government agencies have not established safe exposure limits for these contaminants.

Use Instructions

1. Failure to follow all instructions and limitations on the use of this filter and/or failure to wear the respirator during all times of exposure can reduce respirator effectiveness and **may result in sickness or death.**

2. Before occupational use of these filters, a written respiratory protection program must be implemented meeting all the local applicable requirements. In the U.S., follow OSHA 29 CFR 1910.134 which includes medical evaluation, training and fit testing. In the U.S., users must also comply with applicable OSHA substance specific standards. In Canada, CSA standard Z94.4 requirements must be met and/or requirements of the applicable jurisdiction, as appropriate. In Brazil, follow the requirements of the Respiratory Protection Program of the Ministry of Labor.

3. The airborne contaminants which can be dangerous to your health include those so small that you cannot see them.

4. Leave the contaminated area immediately and contact supervisor if dizziness, irritation, or other distress occurs.

5. Store the cartridge/filters and respirator away from contaminated areas when not in use.

6. Dispose of used cartridge/filters in accordance with applicable regulations.

Use Limitations

- This product does not supply oxygen. Do not use in atmospheres containing less than 19.5% oxygen.
 - Do not use when concentrations of contaminants:
 - are immediately dangerous to life or health (IDLH),
 - are unknown,
 - are greater than 10 times the permissible exposure limit (PEL) with half facepiece respirators and full facepiece respirators when qualitatively fit tested,
 - are greater than 50 times the PEL with full facepiece respirators when quantitatively fit tested, or
 - exceed specific local applicable government regulations (such as OSHA standards in the U.S.) or other applicable government regulations, whichever is lower.
- Do not alter, clean (e.g. vacuum, wash, use compressed air), abuse or misuse these filters and/or respirator.
- Do not use with beards, facial hair or other conditions that prevent a good seal between the face and the sealing surface of the respirator.

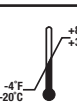
In Brazil, according to the Respiratory Protection Program of the Ministry of Labor, do not use when concentrations of contaminants are greater than 10 times the permissible exposure limit using a half facepiece or 100 times the permissible exposure limit using a full facepiece.

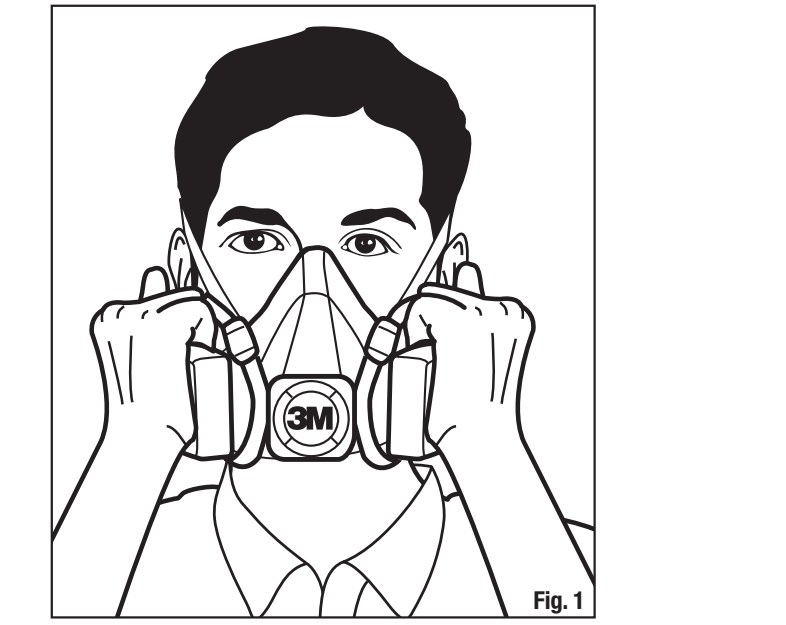
Time Use Limitations

- If any component of the respirator system becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area immediately and replace the filters.
- If used in environments containing only oil aerosols, dispose of filters after 40 hours of use or 30 days, whichever is first.

Storage Conditions

Before use, store product in the original packaging away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture and damaging chemicals. Always inspect product and conduct a user seal check before use as specified in the respirator User Instructions. If you cannot achieve a proper seal, do not enter the contaminated area. See your supervisor.

	Storage Temperature Range	-20°C (-4°F) to +30°C (+86°F)
	Maximum Relative Humidity	<80% RH

Assembly on Facepiece

- Align opening of filter with filter attachment on facepiece. **NOTE:** The alignment mark on the filter should face the front of the facepiece.
- Turn filter clockwise one quarter turn until it is firmly seated and cannot be further turned.
- Repeat for second filter.
- When using the 7093 filters with the 6000 Series Cartridges, the 502 adaptor is required.

NEGATIVE PRESSURE USER SEAL CHECK

1. Using hands press or squeeze filter covers toward facepiece to restrict airflow. Be careful not to disturb the position of the respirator. *Inhale gently.* If you feel the facepiece collapse slightly and pull closer to your face with no leaks between your face and the facepiece, a proper seal has been obtained (Fig. 1).

2. If faceaeal air leakage is detected, reposition respirator on your face and/or readjust tension of straps to eliminate the leakage.

IMPORTANT: If you CANNOT achieve a proper seal, DO NOT enter the contaminated area. See your supervisor.

NIOSH Approved: P100

All but 99.97% filtration efficiency against solid and liquid aerosols including oils. This filter contains no components made from natural rubber latex.

For Compliance in Brazil Note:

PARTICULATE FILTER 7093, P3 SL
In Brazil, Ministry of Labor approved as filter class P3, SL.

NOTE:

- In Brazil do not use when concentrations of contaminants are greater than 100 times the permissible exposure limit in air-purifying mode using full facepiece.
- Do not use in deficient or enriched oxygen atmospheres.
- Storage, Transportation and Care: store in a clean and dry place and away from contaminants and extreme temperature and humidity.
- The components of this respirator are made of materials which are not expected to cause adverse health effects.
- It is necessary to have special care to use this product in explosives atmospheres.

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