Type MA

Biomedical Chip NTC Thermistors

Thermometrics Biomedical Chip Thermistor Assemblies are designed for use in applications involving both intermittent and continuous patient temperature monitoring. Repeatability and fast



response are essential, not only for the intermittent temperature requirements associated with oral and rectal fever measurements, but also for the continuous monitoring often necessary during induced-hypothermia and general anesthesia, or when employed in the care of infants and premature babies. Intensive care units, along with recovery rooms, have also adopted patient temperature as part of their vital sign monitoring procedures. Temperature monitoring for skin surface, tympanic, esophageal, foley catheters, and biofeedback applications has also improved due to the high stability and tight interchangeable tolerances designed into each Thermometrics' Biomedical Sensor Assembly.

Within our Type MA line of NTC Thermistors, we offer a complete line of standard sub-assemblies. Continuous research and development efforts within Thermometrics have resulted not only in these field-proven designs, but in the development of our Unitherm Thermochip Thermistor, designed exclusively for biomedical applications in the range of 32°F to 122°F (0°C to 50°C).

Although low in cost, these highly stable, precision thermochips provide the reliability, tight interchangeable tolerances, geometries, and fast response times that are often required in medical applications.

Thermometrics employs experienced applications engineers who welcome inquiries, whether for assistance, or to help with your design requirements concerning new and existing applications.


Applications

- Intermittent Patient Temperature Monitoring For oral and rectal fever measurements
- Continuous Patient Temperature Monitoring During general anesthesia, induced hypothermia, and premature infant care
- Vital Sign Monitoring For Intensive Care Units (ICU) and recovery rooms
- Temperature Monitoring For skin surface, tympanic, esophageal, foley catheters, and biofeedback



Type MA Specifications

MA100

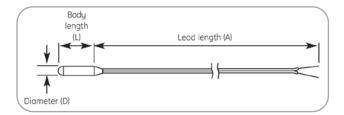


Designed for use in catheter assemblies, these sensors are available with nominal resistance values of 2252, 3000, 5000, and 10,000 Ω at 77°F (25°C).

Close monitoring of manufacturing processes allow Thermometrics to maintain tight interchangeability tolerances with volume production.

Typical design parameters are represented in Assembly Type table.


- 0.030 in (0.76 mm) (10 KΩ only) and 0.050 in (1.27mm) diameters available with kapton sleeve only.
- 0.070 in (1.78 mm) diameter normally supplied with molded plastic tip.

MA200

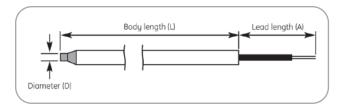

Intermittent body temperature measurements are common practice in all phases of patient care. This assembly is ideally suited for the disposable cover oral and rectal fever thermometers in use today. It features rugged construction with tip sensitive shaft assemblies exhibiting resistance values of 2252, 3000, 5000, and $10,000~\Omega$ at 77° F (25°C). Refer to Assembly Type table for typical design parameters.

MA300

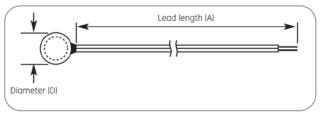

Routine continuous patient temperature monitoring is now feasible by using the convenience of the patient's skin site as an indicator of body temperature. The stainless steel housing used is suitable for both reusable and disposable applications, while maintaining maximum patient comfort. Nominal resistance values of 2252, 3000, 5000, and $10,000~\Omega$ at 77° F (25°C) are available. Refer to Assembly Type table for typical design parameters.

Assembly Type

Assem- bly Type	MA100	MA200	MA300
Standard Diameters (D)	0.030 in (0.762 mm) 0.050 in (1.27 mm) 0.070 in (1.78 mm) 0.080 in (2.03 mm)	0.156 in (3.96 mm)	0.375 in (9.52 mm)
Body Length (L)	3/8 in (9.52 mm)	3.75 in (95.25 mm)	N/A
Standard Lead Length (A)	24 in (610mm) GG: 36 in (914mm)	2 in (51 mm)	24 in (610mm)
Tolerance	See Tolerance Code and Tem- perature Table	See Tolerance Code and Tem- perature Table	See Tolerance Code and Tem- perature Table
Wire Gauge	28, 30, 32, 38, AWG	30 AWG	30 AWG
Standard Wire Insulation	*Heavy Isomid *Medical Grade PVC *Polyurethane with Nylon Overcoat	PTFE	Medical Grade PVC PTFE
Body Material	Molded Plastic or Kapton Sleeve	Lexan Shaft Aluminum Tip	Stainless Steel
Nominal R Values @ 77°F (25°C)		2252 Ω , 3000 Ω , 5000 Ω , 10,000 Ω (for all three types)	

Thermal Response Time (63% Response)

Series	Still Air	Still Water*
MA100 Catherter Assembly	15 seconds	2.0 seconds
MA200 Oral-Rectal Assembly	35 seconds	0.6 seconds
MA300 Skin Surface Assembly	45 seconds	2.0 seconds

*Response time provided is for assembly plunged from 77°F (25°C) air to 41°F (5°C) water.

Type MA Specifications (Cont.)

Tolerance Code and Temperature Range

Tolerance Code

Temperature Range °F (°C)	A ±°F (±°C)	B ±°F (±°C)	C ±°F (±°C)
32 to 68 (0 to 20)	0.27 (0.15)	0.36 (0.2)	0.45 (0.25)
68 to 95 (20 to 35)	0.18 (0.1)	0.27 (0.15)	0.36 (0.2)
95 to 102 (35 to 39)	0.09 (0.05)	0.18 (0.1)	0.27 (0.15)
102 to 107.60 (39 to 42)	0.14 (0.075	0.27 (0.15)	0.36 (0.2)
107.60 to 113 (42 to 45)	0.18 (0.1)	0.27 (0.15)	0.36 (0.2)
113 to 122 (45 to 50)	0.27 (0.15)	0.36 (0.2)	0.45 (0.25)

Data

Biomedical Chip Thermistors and sub-assemblies are designed to be interchangeable over a 32°F to 122°F (0°C to 50°C) range. Best overall stability is maintained when exposure and storage temperatures remain below 158°F (70°C).

Resistance vs Temperature

Temperature °F (°C)	2252 Ω	3 k Ω	5 k Ω	10 k Ω	103(Y) Ω
32.00 (0)	7373.00	9821.93	16369.9	32739.8	29565.8
33.80 (1)	7005.80	9332.76	15554.6	31109.2	28224.8
35.60 (2)	6659.04	8870.84	14784.7	29569.5	26951.9
37.40 (3)	6331.49	8434.49	14057.5	28115.0	25743.3
39.20 (4)	6021.97	8022.17	13370.3	26740.6	24595.5
41.00 (5)	5729.40	7632.41	12720.7	25441.4	23505.0
42.80 (6)	5452.74	7263.87	12106.4	24212.9	22468.7
44.60 (7)	5191.06	6915.27	11525.4	23050.9	21483.8
46.40 (8)	4943.46	6585.43	10975.7	21951.4	20547.2
48.20 (9)	4709.11	6273.23	10455.4	20910.8	19656.6
50.00 (10)	4487.22	5977.64	9962.74	19925.5	18809.3
51.80 (11)	4277.07	5697.69	9496.15	18992.3	18003.1
53.60 (12)	4077.97	5432.47	9054.11	18108.2	17235.7
55.40 (13)	3889.29	5181.11	8635.19	17270.4	16505.1
57.20 (14)	3710.42	4942.83	8238.05	16476.1	15809.4
59.00 (15)	3540.80	4716.88	7861.46	15722.9	15146.7
60.80 (16)	3379.91	4502.54	7504.24	15008.5	14515.3
62.60 (17)	3227.24	4299.17	7165.28	14330.6	13913.6
64.40 (18)	3082.34	4106.14	6843.57	13687.1	13340.0
66.20 (19)	2944.77	3922.88	6538.13	13076.3	12793.0
68.00 (20)	2814.12	3748.83	6248.05	12496.1	12271.4
69.80 (21)	2690.01	3583.49	5972.48	11945.0	11773.8
71.60 (22)	2572.07	3426.38	5710.63	11421.3	11299.0
73.40 (23)	2459.96	3277.03	5461.72	10923.4	10845.8
75.20 (24)	2353.37	3135.04	5225.07	10450.1	10413.1
77.00 (25)	2252.00	3000.00	5000.00	10000.0	10000.0
78.80 (26)	2155.56	2871.53	4785.88	9571.77	9605.42
80.60 (27)	2063.79	2749.28	4582.13	9164.26	9228.45
82.40 (28)	1976.44	2632.91	4388.19	8776.38	8868.24
84.20 (29)	1893.27	2522.12	4203.54	8407.07	8523.96
86.00 (30)	1814.07	2416.61	4027.68	8055.35	8194.82
87.80 (31)	1738.61	2316.09	3860.15	7720.30	7880.09
89.60 (32)	1666.71	2220.31	3700.51	7401.03	7579.07
91.40 (33)	1598.18	2129.02	3548.36	7096.72	7291.10
93.20 (34)	1532.85	2041.98	3403.30	6806.60	7015.55
95.00 (35)	1470.54	1958.98	3264.97	6529.94	6751.83
96.80 (36)	1411.11	1879.81	3133.02	6266.03	6499.37
98.60 (37)	1354.41	1804.27	3007.12	6014.23	6257.64
100.40 (38)	1300.29	1732.18	2886.96	5773.93	6026.13
102.20 (39)	1248.63	1663.36	2772.27	5544.53	5804.36
104.00 (40)	1199.30	1597.65	2662.75	5325.50	5591.88
105.80 (41)	1152.19	1534.89	2558.15	5116.30	5388.26
107.60 (42)	1107.19	1474.94	2458.23	4916.46	5193.09
109.40 (43)	1064.18	1417.65	2362.75	4725.49	5005.96
111.20 (44)	1023.08	1362.89	2271.49	4542.98	4826.53
113.00 (45)	983.78	1310.55	2184.24	4368.49	4654.43
114.80 (46)	946.21	1260.49	2100.82	4201.64	4489.33
116.60 (47)	910.27	1212.62	2021.03	4042.05	4330.91
118.40 (48)	875.89	1166.81	1944.69	3889.38	4178.88
120.20 (49)	842.99	1122.99	1871.65	3743.29	4032.94
122.00 (50)	811.50	1081.04	1801.73	3603.46	3892.83
\ -/					

Type MA Specifications (Cont.)

Ordering Information

The code number to be ordered may be specified as follows:

MA					
1	Code	Standard A	ssemblie	s	
	100FA	Series 100, 0.070 in (1.78 mm) diameter molded plastic tip,			
		30 gauge P'	VC insulat	ed ribbon cable.	
	100FD	Series 100,	0.070 in (⁻	1.78 mm) diameter molded plastic tip,	
		32 gauge bifilar heavy isomid insulation.			
	100DD	Series 100, 0.050 in (1.27 mm) diameter kapton sleeve with			
		32 gauge bifilar heavy isomid insulation.			
	100BF		•	0.768 mm) diameter kapton sleeve with	
				isomid insulation.	
	200LC		•	3.96 mm) diameter aluminum tip,	
		30 gauge P			
	300TA			9.52 mm) diameter stainless steel cup,	
	00075	• •		ed ribbon cable.	
	300TB	Series 300, 0.375 in (9.52 mm) diameter stainless steel cup,			
	10000	30 gauge PTFE insulated ribbon cable.			
	100GG	Series 100, 0.080 in (2.03 mm) diameter molded plastic tip,			
		28 gauge kynar insulated twisted pair.			
		Code		nce Cable*	
		232 302	2252Ω 3000Ω		
		502 502	5000Ω		
		103	$10,000 \Omega$		
		103 103 (Y)	10,000 Ω		
		103 (1)	Code	Tolerance	
			X	Refer to tolerance code and temperature	
			Î	range table	
\downarrow	V	V	\	Tango tabio	
MA -			T	ypical Model Number	

^{*}Refer to resistance vs temperature table on previous page.



X-ON Electronics

Largest Supplier of Electrical and Electronic Components

Click to view similar products for NTC (Negative Temperature Coefficient) Thermistors category:

Click to view products by Amphenol manufacturer:

Other Similar products are found below:

118-253FAJ-P01 121-202EAC-P01 123-802EAJ-P01 128-105NDP-Q02 129-202VME-S01 135-503LAD-J01 B57250V2473F560

B57620C472K962 B57620C5103J062 B57621C5102J062 A1004SG22P0 192-103LPR-A01 199-303KAF-A02 30054-4 B57471V2474H062

B57620C5102J062 B57620C5223J062 500-52AA04-101 526-31AA19-104 526-31AN12-202 103AT-5-1P-FT 10K3A542I 111-802EAJ-901

112-103FAG-H02 112-104KAG-B01 11028414-00 111-182CAG-H01 112-103FAF-H01 112-104KBF-F01 118-202CAJ-P01 526-31AA79-102 B57442V5103J62 517-59CL01-202 B57401V2103H62 B57621C5472J62 11032565-00 194303KEVA01 NTCACAPE3C90193

B57621C5472K062 135-105QAF-J02 B57421V2153J062 B57230V2103H260 B57471V2684H062 B57471V2333H062